

A blue-tinted photograph of a medical training setup. In the foreground, a woman with glasses and a white lab coat is looking at a tablet. The tablet displays a circular diagram with a blue line and several small circles. In the background, a man is wearing a white head-mounted display (HMD) with the 'OPTOMED' logo on it. The entire image has a blue overlay.

OPTOMED

Optomed Polarix Training Deck

Contents

- Setup
 - Connections
 - PACS
 - Auto-export
- Usage
 - General
 - Patient data
 - Capturing
 - Retina
 - Cornea
 - Review
 - Patient data layout
 - Image handling options
 - Integrated viewer




Setup

- Connections
 - The Polaris camera can be connected to a LAN-network with the Ethernet connector in the bottom of the camera.
 - Also USB mass storage and HDMI connectors can be found in the bottom



Setup


- DICOM
 - PACS and Worklist connections can be established by filling out the information in DICOM SETTING in Settings.
 - In case DICOM is not found in Settings it can be enabled by installing a DICOM patch (available from Optomed)
 - Note : Images only appear in PACS after they are sent to DICOM in REVIEW section

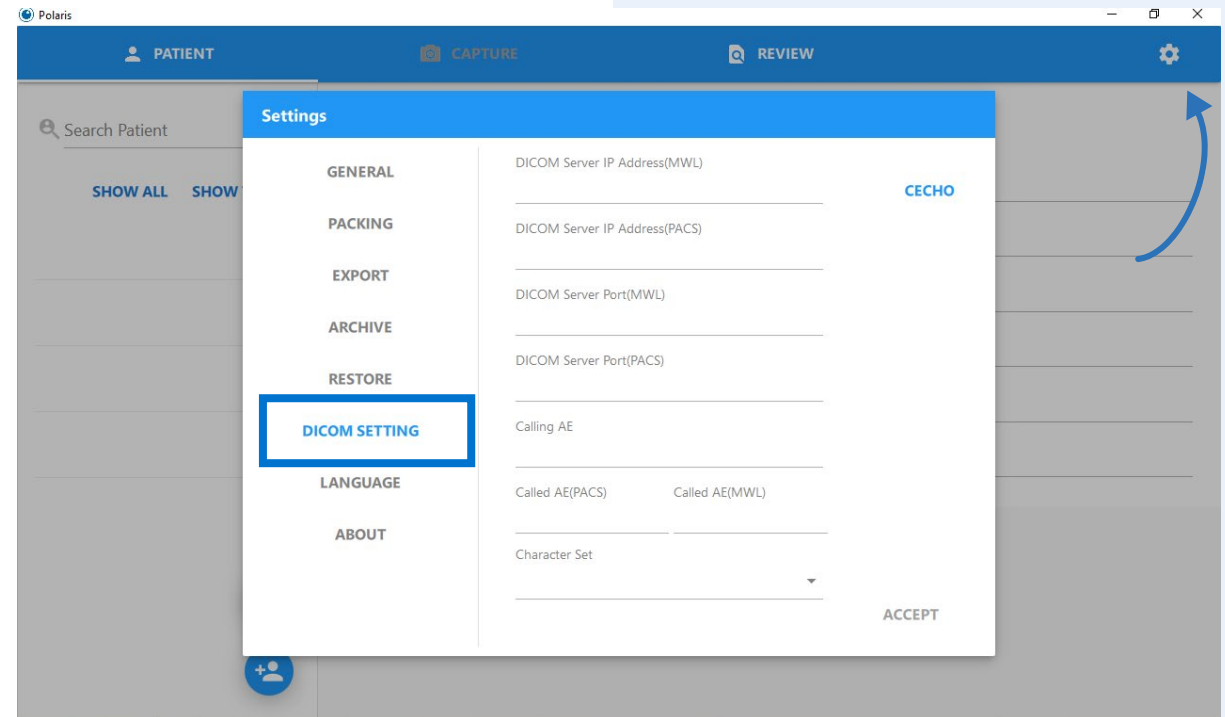



Setup

- Exporting data to a another PC
 - In addition to saving the studies in PACS it is also possible to auto-export all images to a network location using the cameras ethernet connection
 - This can be done by for example mapping a network drive from the built in Windows tablet of Polaris and then defining a target folder for the images in Polaris settings → Export
 - This way image data will be automatically backed up in addition to storing to the Polaris camera

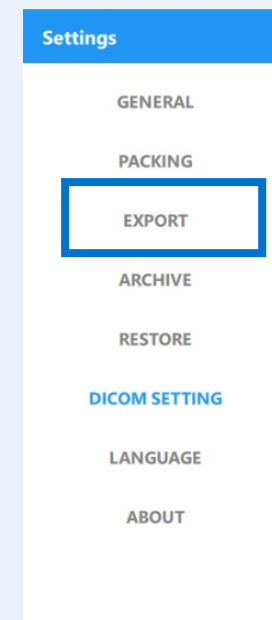

OPTOMED

Usage - General

- Camera Start-up
 - The camera can be started from the I/O switch
 - After the system has powered the Polaris program will open automatically
- Shutting down
 - The camera can be shut down from the I/O switch

Note: Do not close the Polaris program before shutting down



Usage - Patient Data

- Patient data can be filled in the section of UI shown below.
 - Alternatively data can be selected from existing list of patient studies
- To input data select either Manual input or DICOM worklist (Icon is shown if connection is configured)
- Note: ID is a required field

The screenshot shows the 'Polaris' application window with a 'PATIENT' tab selected. The interface is divided into a left sidebar and a main content area. The sidebar contains a search bar labeled 'Search Patient', two buttons 'SHOW ALL' and 'SHOW TODAY', and a section for 'First Lastname' with '0 image(s)'. The main content area has a blue header with 'CAPTURE' and 'REVIEW' tabs. Below the header, it says 'NO PATIENT'. There are several input fields: '*Patient ID' (with 'EMRID' to its right), 'Name', 'Birthday(yyyy-MM-dd)', 'Gender' (a dropdown menu), 'Age', 'Phone', 'Language' (a dropdown menu), 'Email', 'Address', and 'Comment'. At the bottom of the sidebar, there is a blue box containing two circular icons: the top one has a magnifying glass and 'DCM' (representing DICOM worklist), and the bottom one has a plus sign and a person icon (representing manual input). The main content area at the bottom is a large grey rectangle with the text 'NO IMAGE'.

Usage - Patient Data


- Patient data is stored in the local database of the camera (even if PACS connection is used)
- Removal
 - Individual Patient data can be removed by clicking on the Patient data and selecting Remove (1)
 - The whole database in the camera can be erased by using a separate service program, please contact Optomed for details.

The screenshot shows the Polaris software interface. At the top is a blue navigation bar with 'PATIENT', 'CAPTURE', and 'REVIEW' tabs. Below this is a search bar labeled 'Search Patient' with 'SHOW ALL' and 'SHOW TODAY' buttons. A patient entry for 'polarist^test' is selected, showing '0 image(s)'. To the right, the patient's details are displayed: Name 'polarist^test', Birthdate '2020-09-03', Gender 'Female', Language 'English', and fields for Age, Phone, Address, and Comment. A red trash icon is circled in blue with the number '1' next to it, indicating the removal option. At the bottom right, a large grey area contains the text 'NO IMAGE'. The Optomed logo is visible at the bottom center of the slide.

Usage - Capturing

Retina




- Preparations
 - Remove the lens cap
 - Clean the chin- and forehead rests
 - Ask the patient to position their head in the chin rest
- Retinal imaging Operation modes
 - The camera can operate in either
 - Semi-Auto mode which captures only the selected eye
 - Full-Auto mode which captures both eyes sequentially
 - Selecting between the modes can be done in Advanced settings in Capture section of UI

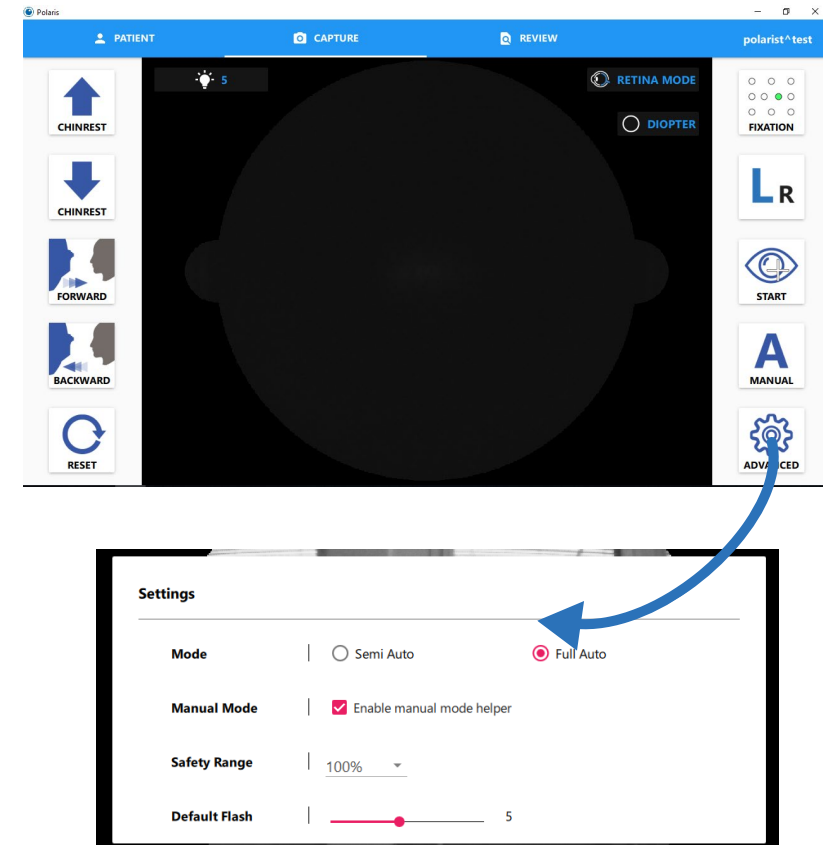

Usage - Capturing

Retina, semi-auto mode



1. First select CAPTURE section from UI
2. Select eye side – and fixation target (if needed)
3. Adjust chin rest height
4. Click center of the pupil
5. Start capture

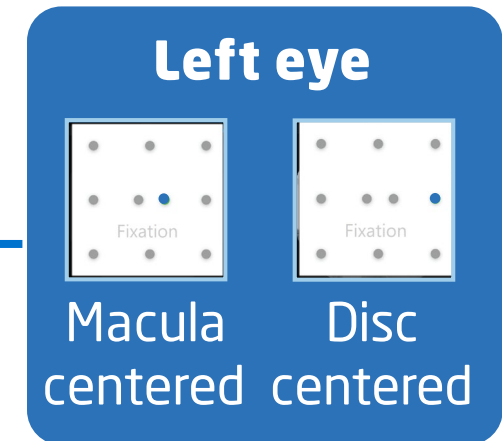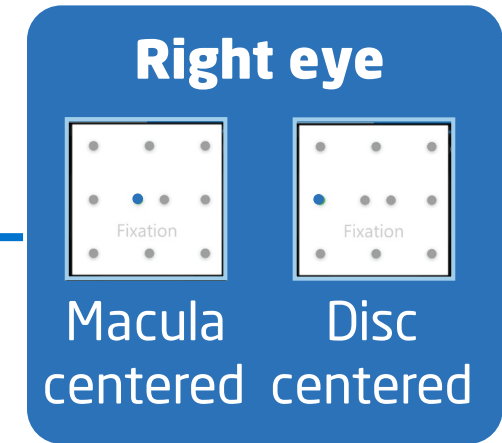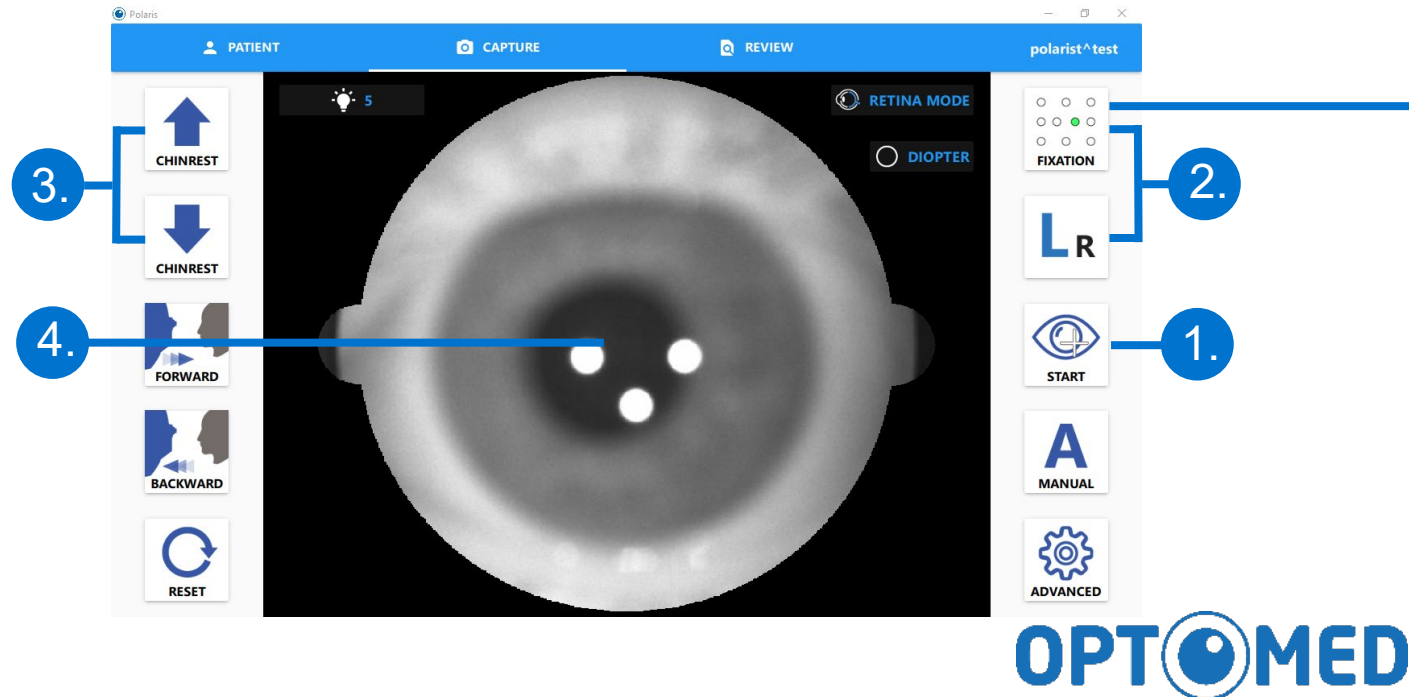


Usage - Capturing

Retina, Full-auto mode


- Macula centered images of both eyes
 - Full-auto mode will automatically capture macula centered images of both eyes with default setting by just adjusting the chin rest height, centering the pupil by clicking on screen and pressing “Capture”
- 4 image sequence
 - A sequence of Macula- and Disc-centered images of both eyes can be captured in the following way
 - Select the correct Fixation target for **Disc image of the eye to be imaged**
 - Capture image : A **disc centered** image of the first eye and **macula centered** of the second eye are captured
 - Repeat the procedure for the second eye – similar pattern will follow

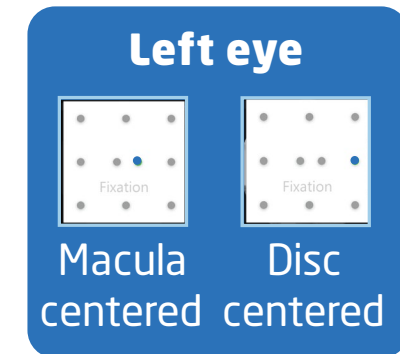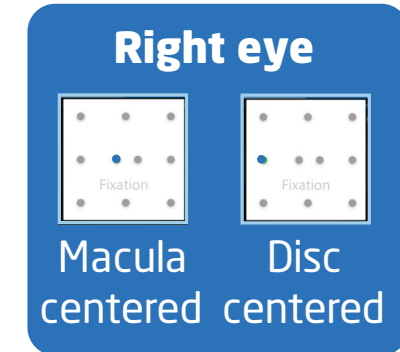

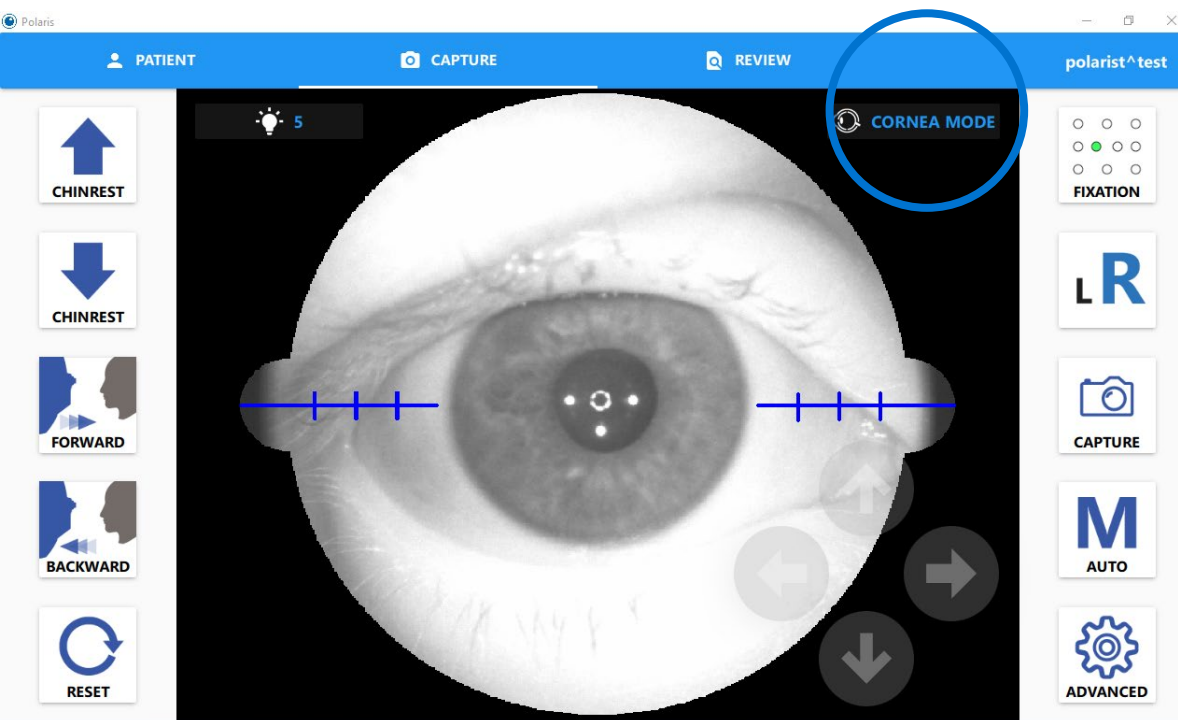

Usage - Capturing Cornea


- Cornea mode can be enabled by clicking the Retina / Cornea icon
- Select eye side, adjust chin rest, click center of pupil and Capture

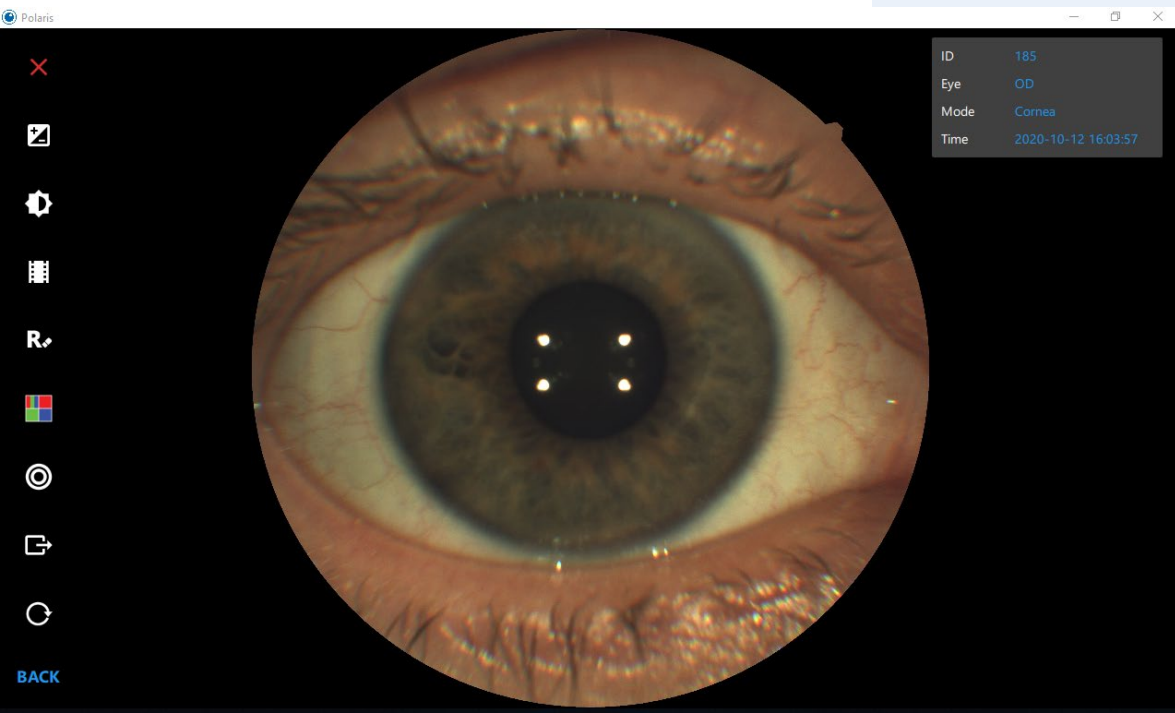

OPTOMED


Usage - Image review

- Go to REVIEW section of UI
- Select patient and Visit Date from list to access image directory

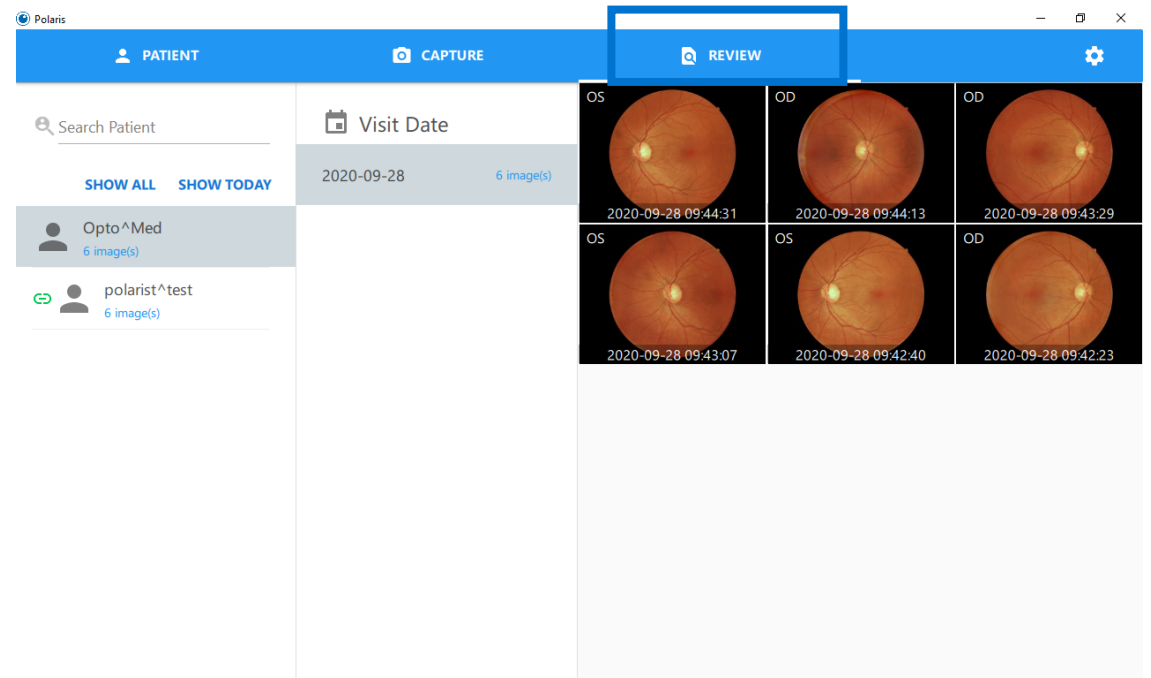

Usage - Image review

- Long press on each image will open handling options (send to DICOM, Mosaic, Print, Side by side comparison, Export to local folder, Delete)
 - Note : Images only appear in PACS after they are sent to DICOM in REVIEW section

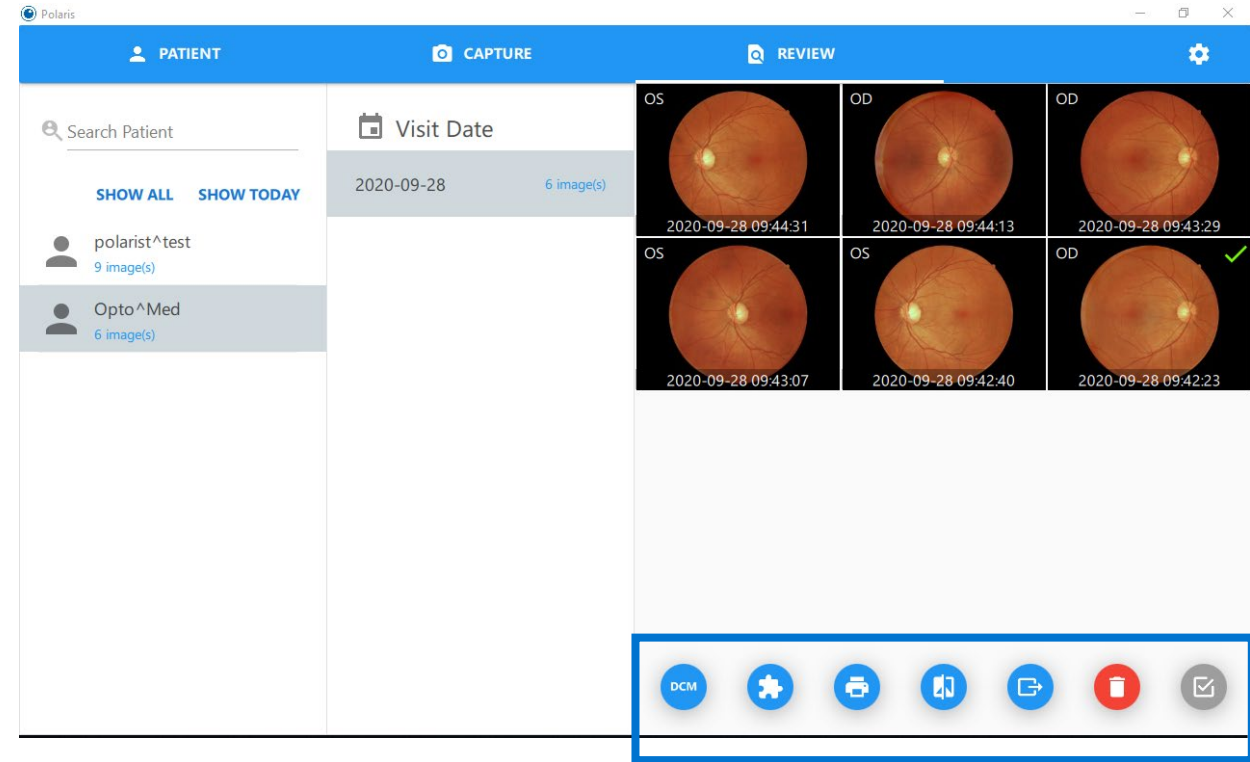

Usage - Image review

- Click on each image will open the viewer with the following features
 - Pinch zoom
 - Contrast adjustment
 - Brightness adjustment
 - Negative
 - Digital Red free
 - Color channel split
 - Cup-to-disc ratio measurement
 - Export
 - Undo



Optomed is a Valued
Supplier on the IAPB
Standard List



OPTOMED

CORPORATION

Optomed Plc. Yrttpellontie 1, 90230 Oulu, Finland
Tel: +358 20 741 3380 • Info@optomed.com
www.optomed.com

