


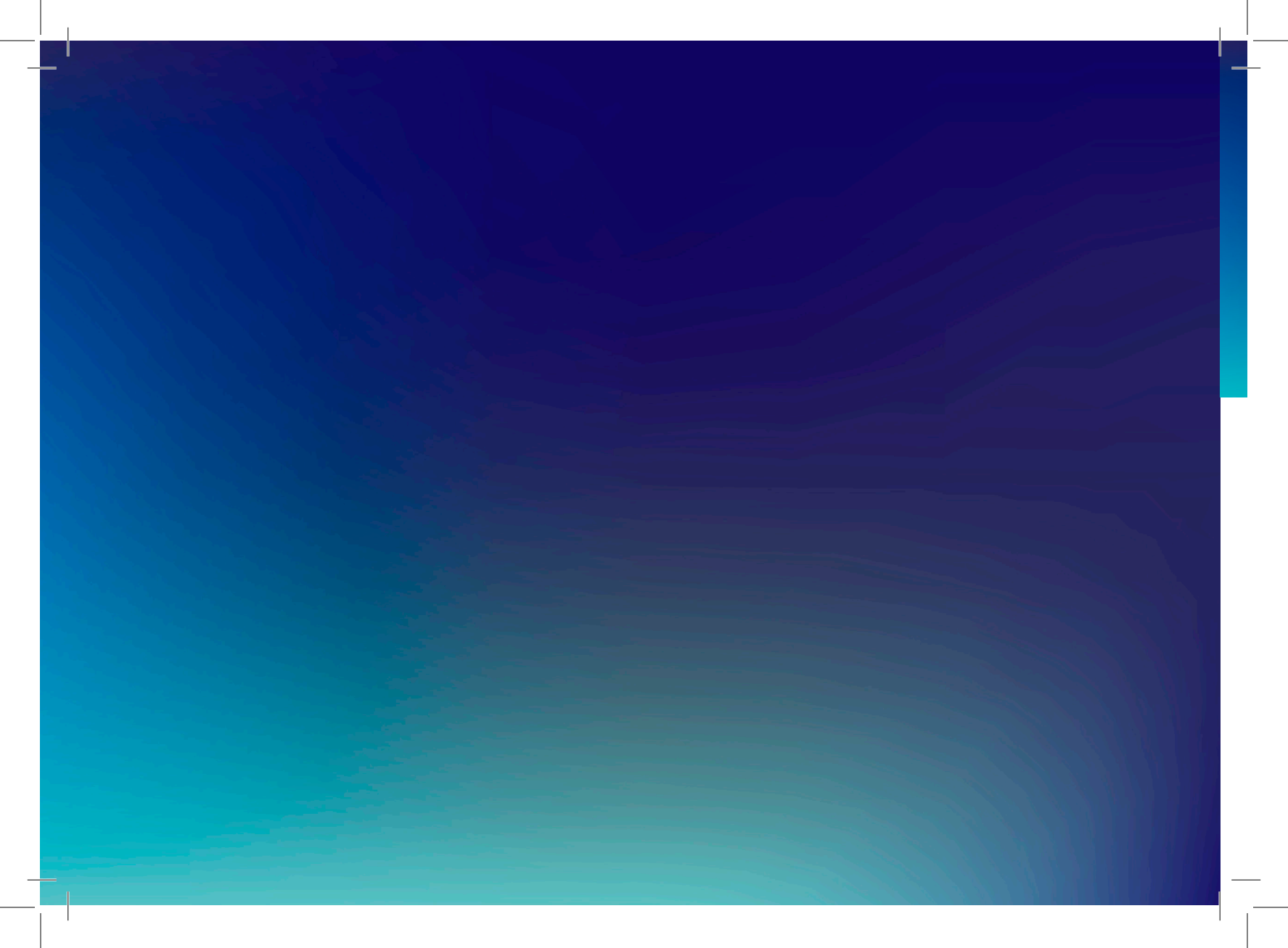

OPTOMED AURORA A EYE

Quick Start Guide

OPTOMED x A EYE Health



This Quick Start Guide is for initial Portal and Camera setup and configuration only.

1.

Preparations

2.


Portal first time use

3.

Camera basic configuration


OPTOMED AURORA **AEYE** CAMERA Preparations

1.

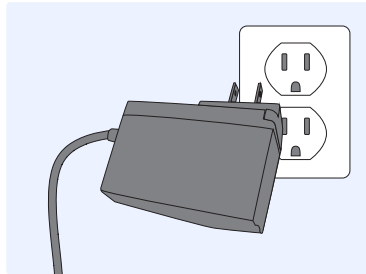


Connect power cord
Connect power supply cable to the mains

2.

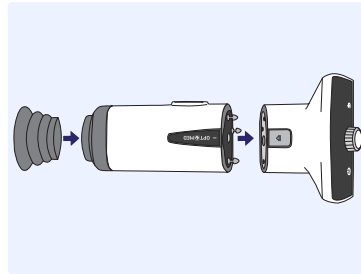


Attach Retinal Module with Eye Cup to Camera

3.

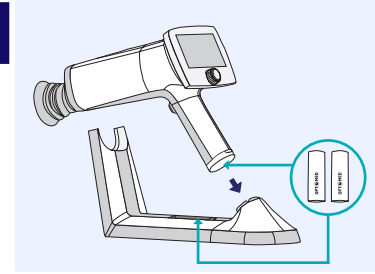


Insert batteries and charge Camera on Charging Station

OPTOMED AURORA AEEYE PORTAL First time use

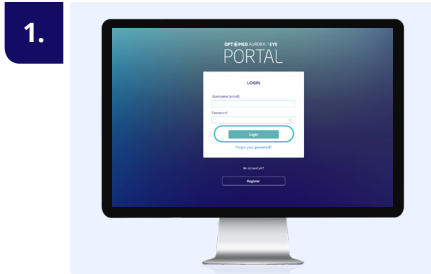

1. Access Optomed Aurora AEEYE Portal at <https://auroraeye.optomed.com>

- Enter username and password
- Select Login

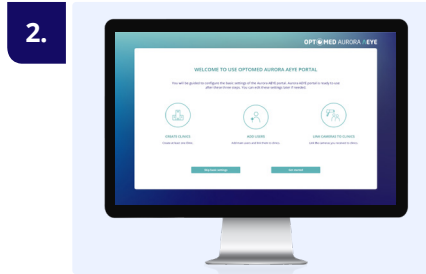


2. Single user:

- Select Skip basic settings, Portal ready


Multiple users:

- Select Getting started, go to step 3

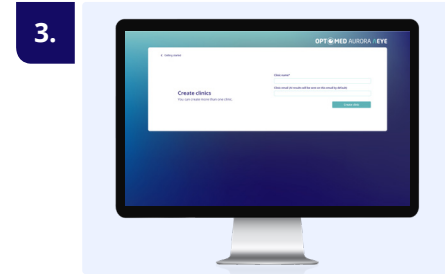


3. Select Add new clinic

- Enter clinic information
- Select Create clinic
- Select Next when all clinics are created

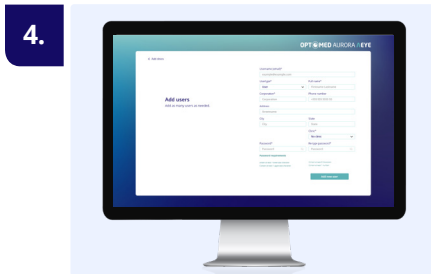


4. Select Add new user

- Enter user information
- Select Add new user
- Select Next when all users are created

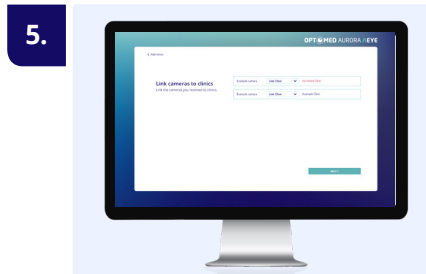

5. Link cameras to clinics

- Select clinic for camera
- Select Next
- Select Next to enter Portal

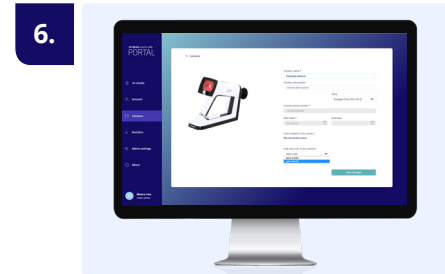


6. Link cameras to users


- Select Cameras tab
- Select Camera > Open
- Select User > Save changes, Portal ready

Continue to set up Optomed Aurora AEEYE Camera software on the next page when Portal is ready


OPTOMED AURORA AEYE CAMERA Basic configuration

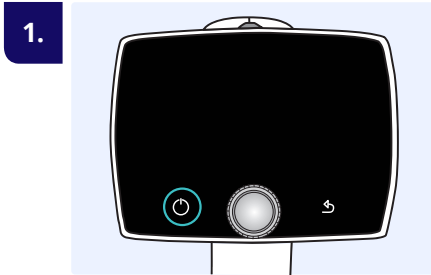


Press Power button

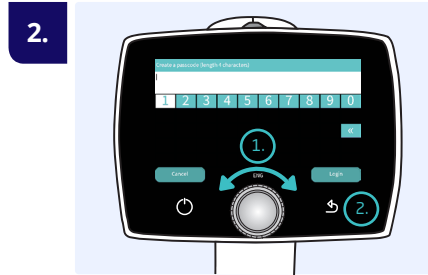


Create passcode for Admin;
choose 4 digits*
1. Optoroller; rotate and press
2. Back button; press to Login

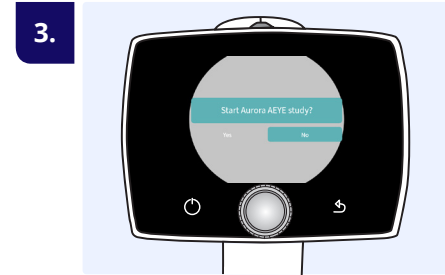


Select No for Start Aurora AEYE
study query; Live view opens

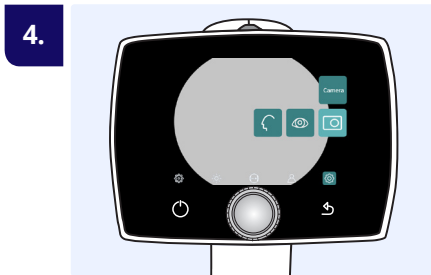

Enter menu by pressing Optoroller *
Set up wireless connection;
Settings > Camera >

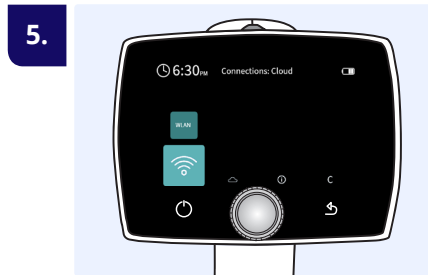

> Connections > WLAN
• Select WLAN connection; On
• Network; select available wireless
network from the list

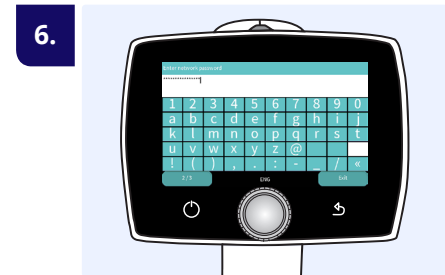

Enter network password;
Press Power button to access
lowercase and more characters
• Select Exit


*To navigate between items; rotate Optoroller to the left or right, press Optoroller to select the item. Login/Exit/Next is selected by pressing Back button.

7.



Select Connect

8.

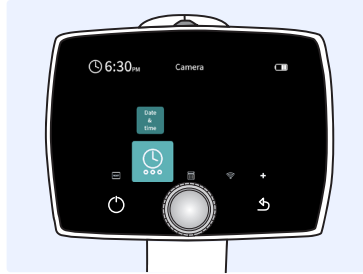

Set up Date & Time

- Daylight saving time; if applicable
- Time zone; choose AST, EST, CST, MST, PST, AKST, HST, SST, or CHST

9.



Automatic time sync


- Select On

10.



Select Sync now

11.

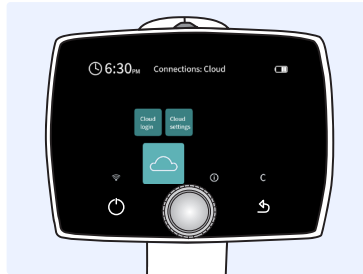

Press Back button to enter Cloud connections. Select Cloud login.

12.



Enter Cloud username (same as for Optomed Aurora AEYE Portal). Select Exit.


13.



Enter Cloud password


- Select Exit
- Select Login

14.

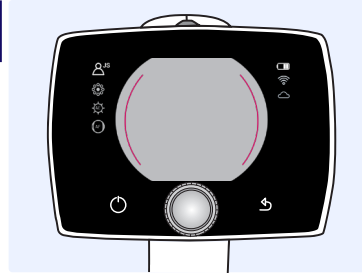

Press Back button to Exit the settings, until in Live view

15.

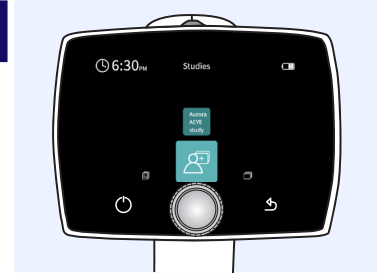

Optomed Aurora AEYE Camera is ready for Aurora AEYE study



Please refer to Optomed Aurora AEYE Quick Guide and User Manual next for imaging technique and workflow

R5.0
English 60002880

Optomed USA
support.us@optomed.com
www.optomed.com/us

OPTOMED x  **A EYE Health**